## **ENDOCAPSULAR TENSION RING**

An Endocapsular tension ring is indicated to be used for circular expansion and stabilization of the capsular bag in an ophthalmic surgical procedure.

### Instruction for use:

- Open the pouch of sterile Endocapsular tension ring (BY AN UNSTERILE PERSON) and drop the inner container with Endocapsular tension ring on a sterile surface without touching it.
- Open the container cap and take out the Endocapsular tension ring and wash it with saline water
- Feed one eyelet of the ring into the capsular bag using a Mcpherson forcep or using an Endocapsular tension ring injector.
- Then slowly insert the ring in capsular bag using the tension ring injector or a lens dialor till only the second eyelet remains outside the eye
- Insert the second eyelet of the ring in the bag using a McPherson forcep and
- check the capsular bag integrity.

  Proceed further to complete the remaining Phaco or IOL inserting procedure.
- The empty container is put back into the pouch and sent with other waste for incineration.
- For single use only.
- Store it in a cool and dry place
- Do not expose to heat or direct sunlight.

  The product should be used immediately once the packing is opened.
- Device is to be used by qualified doctors/competent persons.

  The product should be handled according to the norms.

  If it is reused, then it may cause serious problems to patients such as HIV, Hepatitis, contagious dieses/communicable disease/undue diseases etc

# **⚠** Warning

- Read instruction before use.
- The products should not be re-sterilized once used, must inspect and check the product before use
- Should not be used if the packing is damaged.
- Mishandling or improper transport can cause damage to the device or
- . The product is non-toxic, sterile and non-pyrogenic if the packaging is intact.

### Storage:

Store at a temperature between 10°C to 40°C and a relative humidity less than 65% Do not expose to direct sunlight. Expiry date on the pack indicates the period within which the device should be used.

### Contraindications:

- If it is used for any other purposes than indicated use, will be the responsibility of the user.
- If product is used by uncompetent/untrained personel



REF TE/ART/2015-16/01 L. Rev. No. 06 dated 24th May 2022

# Ophthalmic Surgical Instruments-

## **ENDOCAPSULAR TENSION RING**

### Indication:

An Endocapsular tension ring is indicated to be used for circular expansion and stabilization of the capsular bag in an ophthalmic surgical procedure.

### Instruction for use:

- Open the pouch of sterile Endocapsular tension ring (BY AN UNSTERILE PERSON) and drop the inner container with Endocapsular tension ring on a sterile surface without touching it.
- Open the container cap and take out the Endocapsular tension ring and wash it with saline water.
- Feed one eyelet of the ring into the capsular bag using a Mcpherson forcep or using an Endocapsular tension ring injector.

  Then slowly insert the ring in capsular bag using the tension ring injector or a lens dialor till only the second eyelet remains outside the eye.

  Insert the second eyelet of the ring in the bag using a McPherson forcep and check the capsular bag intority.
- check the capsular bag integrity. Proceed further to complete the remaining Phaco or IOL inserting procedure.
- The empty container is put back into the pouch and sent with other waste for incineration.
- For single use only. Store it in a cool and dry place.
- Do not expose to heat or direct sunlight.
  The product should be used immediately once the packing is opened
- Device is to be used by qualified doctors/competent persons The product should be handled according to the norms.
- If it is reused, then it may cause serious problems to patients such as HIV, Hepatitis, contagious dieses/communicable disease/undue diseases etc

# **⚠** Warning

- Read instruction before use
- The products should not be re-sterilized once used, must inspect and check the product before use
- Should not be used if the packing is damaged.
- Mishandling or improper transport can cause damage to the device or
- The product is non-toxic, sterile and non-pyrogenic if the packaging is intact.

Store at a temperature between 10°C to 40°C and a relative humidity less than 65% Do not expose to direct sunlight. Expiry date on the pack indicates the period within which the device should be used.

Contraindications: If it is used for any other purposes than indicated use, will be the responsibility of the user.

If product is used by uncompetent/untrained personel



/elet remains outside the eye. ring in the bag using a McPherson forcep

not expose to heat or direct sunlight.
product should be used immediately once the packing is opened ice is to be used by qualified doctors/competent persons.
product should be handled according to the norms.

þe Ξ purposes than indicated use, Contraindications:

If it is used for any other purposes than indic responsibility of the user.

If product is used by uncompetent/untrained personel

STERILE

Ophthalmic Surgical Instruments-

Indication:

Instruction for use:

For single use only.

Warning
 Read instruction before use.

which the device should be used.

TARUN ENTERPRISES

+91 9335154556

Ophthalmic Surgical Instruments-

it with saline water.

incineration. For single use only.

• Read instruction before use.

the product before use.

which the device should be used

Contraindications:

Storage:

check the capsular bag integrity.

Store it in a cool and dry place.
Do not expose to heat or direct sunlight.

Should not be used if the packing is damaged.

If product is used by uncompetent/untrained personel

STERILE EO 🌋 🛞

**Regd. Off.:** 8/8, Strachy Road Prayagraj (Allahabad) - 211001, U.P., INDIA

e-mail: info@optitechevecare.com

Works: Shed No. E 20 NSIC Campus, Mirzapur Road Naini, Prayagraj (Allahabad)-211009, U.P., INDIA

REF TE/ART/2015-16/01 L. Rev. No. 06 dated 24th May 2023

**TARUN ENTERPRISES** 

+91 9335154556

the

Indication:

Contraindications:

Storage:

Store it in a cool and dry place.

Do not expose to heat or direct sunlight.

**ENDOCAPSULAR TENSION RING** 

using an Endocapsular tension ring injector.

stabilization of the capsular bag in an ophthalmic surgical procedure.

An Endocapsular tension ring is indicated to be used for circular expansion and

Open the pouch of sterile Endocapsular tension ring (BY AN UNSTERILE PERSON) and drop the inner container with Endocapsular tension ring on a

sterile surface without touching it.

Open the container cap and take out the Endocapsular tension ring and wash

Feed one eyelet of the ring into the capsular bag using a Mcpherson forcep or

Then slowly insert the ring in capsular bag using the tension ring injector or a lens dialor till only the second eyelet remains outside the eye.

Insert the second eyelet of the ring in the bag using a McPherson forcep and check the capsular bag integrity.

Proceed further to complete the remaining Phaco or IOL inserting procedure.

The empty container is put back into the pouch and sent with other waste for

If it is reused, then it may cause serious problems to patients such as HIV, Hepatitis, contagious dieses/communicable disease/undue diseases etc

The products should not be re-sterilized once used, must inspect and check the product before use.

Should not be used if the packing is damaged.

Mishandling or improper transport can cause damage to the device or

The product is non-toxic, sterile and non-pyrogenic if the packaging is intact.

If it is used for any other purposes than indicated use, will be the

STERILE EO SERH OF SERH OF SERH

An Endocapsular tension ring is indicated to be used for circular expansion and

Open the pouch of sterile Endocapsular tension ring (BY AN UNSTERILE PERSON) and drop the inner container with Endocapsular tension ring on a

sterile surface without touching it.

Open the container cap and take out the Endocapsular tension ring and wash

Feed one eyelet of the ring into the capsular bag using a Mcpherson forcep or

using an Endocapsular tension ring injector.
Then slowly insert the ring in capsular bag using the tension ring injector or a lens dialor till only the second eyelet remains outside the eye.

Insert the second eyelet of the ring in the bag using a McPherson forcep and

Proceed further to complete the remaining Phaco or IOL inserting procedure. The empty container is put back into the pouch and sent with other waste for

If it is reused, then it may cause serious problems to patients such as HIV, Hepatitis, contagious dieses/communicable disease/undue diseases etc

The products should not be re-sterilized once used, must inspect and check

Mishandling or improper transport can cause damage to the device or

The product is non-toxic, sterile and non-pyrogenic if the packaging is intact.

If it is used for any other purposes than indicated use, will be the

Store at a temperature between 10°C to 40°C and a relative humidity less than 65%.

Do not expose to direct sunlight. Expiry date on the pack indicates the period within

The product should be used immediately once the packing is opened. Device is to be used by qualified doctors/competent persons. The product should be handled according to the norms.

Store at a temperature between  $10^{\circ}\text{C}$  to  $40^{\circ}\text{C}$  and a relative humidity less than 65%. Do not expose to direct sunlight. Expiry date on the pack indicates the period within

If product is used by uncompetent/untrained personel

**Regd. Off.**: 8/8, Strachy Road Prayagraj (Allahabad) - 211001, U.P., INDIA

e-mail: info@optitecheyecare.com

**ENDOCAPSULAR TENSION RING** 

stabilization of the capsular bag in an ophthalmic surgical procedure

Works: Shed No. E 20 NSIC Campus, Mirzapur Road Naini, Prayagraj (Allahabad)-211009, U.P., INDIA

REF TE/ART/2015-16/01 L. Rev. No. 06 dated 24th May 2022

The product should be used immediately once the packing is opened.

Device is to be used by qualified doctors/competent persons. The product should be handled according to the norms.

Regd. Off.: 8/8, Stract Prayagraj (Allahabad) Works: Shed No. E 2l Naini, Prayagraj (Allah e-mail: info@optiteche

Ophthalmic Surgical Instruments-

## **ENDOCAPSULAR TENSION RING**

An Endocapsular tension ring is indicated to be used for circular expansion and stabilization of the capsular bag in an ophthalmic surgical procedure.

## Instruction for use:

- Open the pouch of sterile Endocapsular tension ring (BY AN UNSTERILE PERSON) and drop the inner container with Endocapsular tension ring on a sterile surface without touching it.
- Open the container cap and take out the Endocapsular tension ring and wash
- it with saline water. Feed one eyelet of the ring into the capsular bag using a Mcpherson forcep or using an Endocapsular tension ring injector.
- Then slowly insert the ring in capsular bag using the tension ring injector or a lens dialor till only the second eyelet remains outside the eye.
- Insert the second eyelet of the ring in the bag using a McPherson forcep and check the capsular bag integrity.
- Proceed further to complete the remaining Phaco or IOL inserting procedure.
- The empty container is put back into the pouch and sent with other waste for incineration.
- For single use only. Store it in a cool and dry place.

- Do not expose to heat or direct sunlight.
  The product should be used immediately once the packing is opened.
- Device is to be used by qualified doctors/competent persor The product should be handled according to the norms.
- If it is reused, then it may cause serious problems to patients such as HIV, Hepatitis, contagious dieses/communicable disease/undue diseases etc

- Read instruction before use.
- The products should not be re-sterilized once used, must inspect and check the product before use.
- Should not be used if the packing is damaged.
- Mishandling or improper transport can cause damage to the device or
- packaging. The product is non-toxic, sterile and non-pyrogenic if the packaging is intact.

# Store at a temperature between 10°C to 40°C and a relative humidity less than 65%.

Do not expose to direct sunlight. Expiry date on the pack indicates the period within which the device should be used Contraindications:

- If it is used for any other purposes than indicated use, will be the responsibility of the user.
  - If product is used by uncompetent/untrained personel



REF TE/ART/2015-16/01 L. Rev. No. 06 dated 24th May 2022

e-mail : info@optitecheyecare.com \$\infty\$ +91 9335154556

### Ophthalmic Surgical Instruments-**ENDOCAPSULAR TENSION RING**

Instruction for use:

An Endocapsular tension ring is indicated to be used for circular expansion and stabilization of the capsular bag in an ophthalmic surgical procedure.

- Open the pouch of sterile Endocapsular tension ring (BY AN UNSTERILE PERSON) and drop the inner container with Endocapsular tension ring on a sterile surface without touching it. Open the container cap and take out the Endocapsular tension ring and wash
- Feed one eyelet of the ring into the capsular bag using a Mcpherson forcep or using an Endocapsular tension ring injector. Then slowly insert the ring in capsular bag using the tension ring injector or a lens dialor till only the second eyelet remains outside the eye.
- Insert the second eyelet of the ring in the bag using a McPherson forcep and check the capsular bag integrity.
- Proceed further to complete the remaining Phaco or IOL inserting procedure. The empty container is put back into the pouch and sent with other waste for
- For single use only Store it in a cool and dry place.
- Do not expose to heat or direct sunlight.
  The product should be used immediately once the packing is opened.
- Device is to be used by qualified doctors/competent persons. The product should be handled according to the norms.
- If it is reused, then it may cause serious problems to patients such as HIV, Hepatitis, contagious dieses/communicable disease/undue diseases etc
- Read instruction before use.
- The products should not be re-sterilized once used, must inspect and check
- the product before use. Should not be used if the packing is damaged. Mishandling or improper transport can cause damage to the device or
- The product is non-toxic, sterile and non-pyrogenic if the packaging is intact.

# Storage:

Store at a temperature between 10°C to 40°C and a relative humidity less than 65%. Do not expose to direct sunlight. Expiry date on the pack indicates the period within which the device should be used.

# Contraindications:

- If it is used for any other purposes than indicated use, will be the responsibility of the user.
  - If product is used by uncompetent/untrained personel



REF TE/ART/2015-16/01 L. Rev. No. 06 dated 24th May 2022

+91 9335154556

Ophthalmic Surgical Instruments-ENDOCAPSULAR TENSION RING

ring is indicated to be used for circular expansion bag in an ophthalmic surgical procedure.

out touching it. cap and take out the Endocapsular