

40mm
40mm

110mm

EXPLANATION OF SYMBOLS

Symbol	Meaning
	Caution
	Manufacturer
	Date of Manufacture
	Do not re-sterilize
	Consult instructions for use
	Keep away from sunlight
	Don't re-use
	Authorized Representative in European Community
	This side up
	Do not use hook
	Conformity of European Norm with Notified Body No
	Batch Code
	Use by date
	Temperature Limit
	Do not use if package is damaged
	Humidity Limitation
	Sterilized using Ethylene Oxide
	Handle with care
	Maximum stacking

TARUN ENTERPRISES
 Works: Shed No. E-20, NSIC Campus
 Mirzapur Road, Naini
 Prayagraj-211009, UP, India
 Regd. Office: 8/8, Strachy Road
 Prayagraj-211001, Uttar Pradesh, India
 Customer Care No.: +91-8176080204
 Email: info@optitecheyecare.com
 Website: www.optitecheyecare.com

Ref. No. TE/IFU/02, Issue No. 01, Issue Date : 12.12.22,
 Rev. No. : 00, Rev. Date : Nil

Rose Bengal
Ophthalmic Strips

INTENDED USE
 A disposable sterile Rose Bengal Strip is used as a vital stain when superficial conjunctival or corneal tissue change is suspected.

DEVICE DESCRIPTION
 A disposable Whatman filter paper No. 1 round blunt tip (impregnated with Rose Bengal dye) sandwiched between a sticker paper stick (overall length 60mm). Each strip is packed individually in pouches before sterilization.

HOW SUPPLIED
 Rose Bengal strips are supplied sterile, in individual pouches. 100 such pouches are enclosed within an outer printed box. The overall packing contains Rose Bengal strips and medical literature insert, in an outer enclosing box, with external identification stickers.

⚠WARNING

- Tarun Enterprises, Ltd. does not hold any responsibility if the device is reused. Reusing the device may cause serious problems to patients such as

FRONT

Page 4
Page 1

40mm
40mm

110mm

HIV, Hepatitis, contagious disease/communicable disease/undue diseases etc.

- Only qualified medical practitioner should use the device.
- Do not use the device if the sterilized package is open or damaged.
- Sterility is guaranteed only when the packing is neither opened nor damaged.
- Do not re-sterilize the device by any method Tarun Enterprises assumes no liability for devices that have been re-sterilized.
- Do not reuse the device.
- Read instructions before use.
- Store at specified temperature and humidity.
- Not to be used with soft contact lenses.
- The used strip and pouch should be disposed off in compliance with the local regulatory requirements.
- Do not use the device after sterility expiry date.
- For External use only.
- Keep out of reach of children.

DIRECTIONS FOR USE

- Moisten Rose Bengal impregnated strip with sterile water or sterile ophthalmic solution.

- Tear fluid from the lower fornix may be used.
- While the patient looks up, touch conjunctiva or fornix with moistened tip.
- For optimal results, the patient should blink several times after application.

EXPIRATION DATE
 The expiration date of the product is the sterility expiration date. The product should not be use after the indicated sterility expiration date.

RETURN OF DAMAGED PRODUCT
 Return the product in its original packing identified by the batch number, purchase information, your reference and reason for return. Please contact your local distributor office regarding product return/exchange.

REPORTING
 Adverse events and/or potentially sight-threatening complications that may reasonably be regarded as product related and that were not previously expected in nature, severity or incidence must be report Tarun Enterprises.

BACK

Page 2
Page 3