

40mm
40mm

EXPLANATION OF SYMBOLS

Symbol	Meaning
	Caution
	Manufacturer
	Date of Manufacture
	Do not re-sterilize
	Consult instructions for use
	Keep away from sunlight
	Don't re-use
	Authorized Representative in European Community
	This side up
	Do not use hook
	Conformity of European Norm with Notified Body No
	Batch Code
	Use by date
	Temperature Limit
	Do not use if package is damaged
	Humidity Limitation
	Sterilized using Ethylene Oxide
	Handle with care
	Maximum stacking

TARUN ENTERPRISES
 Works: Shed No. E 20, NSIC Campus
 Mirzapur Road, Naini
 Prayagraj-211009, UP, India
 Regd. Office: 8/8, Strachy Road
 Prayagraj-211001, Uttar Pradesh, India
 Customer Care No. : +91-8176080204
 Email: info@optitecheyecare.com
 Website: www.optitecheyecare.com

MED DEVICES LIFESCIENCES B.V.
 Keizersgracht 482, 1017EG
 Amsterdam, The Netherlands
 Email: info@meddevices.net
 Phone: +31-202754558

Ref. No. TE/IFU/02, Issue No. 01, Issue Date : 12.12.22,
 Rev. No. : 00, Rev. Date : Nil

Schirmer Tear Test
Ophthalmic Strips

DEVICE DESCRIPTION
 A disposable Whatman Filter Paper No. 41 graduated strip (5mm width x 41mm length) with a least count of 1 mm, having a round blunt tip and is packed individually in pouches before sterilization.

INTENDED USE/INDICATION FOR USE
 A disposable sterile ophthalmic diagnostic Schirmer Strip (External Use Only) is used to measure the amount of tear secretion of the eye.

TARGET AGE GROUP
 Use All Age Groups

DURATION OF USE
 Transient Use (Less than 60 Minutes). It is recommended to withdraw the product after 60 minutes from the patient. The user is accountable for any type of problem taking place in case of use after 60 minutes of the same device.

STORAGE CONDITIONS
 • Store at specified temperature and humidity (Temperature in between 10°C to 40°C and Humidity in between 35RH to 65RH).

CONTRAINDICATIONS
 The device should not be used in patients who show hypersensitivity to any of the components of the preparation. Schirmer Tear Test Strips contraindicated in patients with ocular surface damage, inflammation or ulcers of cornea and conjunctiva.

NATURE OF BODY CONTACT
 Eyeball

MODE OF ACTION
 To conduct a Schirmer's test, a doctor places a piece of filter paper inside the lower eyelid of both eyes and the person closes their eyes. After 5 minutes, the doctor removes the filter paper. The doctor then assesses how far the tears have travelled on the paper. In general, the smaller the amount of moisture on the paper, the fewer tears that person has produced.

HOW SUPPLIED
 Schirmer tear test strips are supplied sterile, in individual pouches. The overall packing contains Schirmer tear test strips and medical literature

110mm
FRONT

Page 4
Page 1

40mm
40mm

insert, in an outer enclosing box, with external identification stickers.

MATERIALS USED
 Whatman Filter Paper No. 41

CAUTIONS
 Do not use the device if the sterilized package is open or damaged.

WARNING

- Tarun Enterprises does not hold any responsibility if the device is reused. Reusing the device may cause serious problems to patients are as under :-
 - The used strip might not have printed scale and may not comply with the claim.
 - The filter paper tip gets deformed after wetting. The used strip might not be in a to be used again due to this deformed shape.
 - Reusing the device may cause the transfer of contagious and communicable disease to the patient and other serious problems to patients due to re-use of device such as HIV, Hepatitis, contagious diseases/communicable disease/undue diseases etc.
- Only qualified medical practitioner should use the device as per the intended use.
- Sterility is guaranteed only when the packing is neither opened nor damaged.
- Do not re-sterilize the device by any method. Tarun Enterprises assumes no liability for devices that have been re-sterilized.
- Do not reuse the device.
- Read instruction before use.
- Store at specified temperature and humidity (Temperature in between 10°C to 40°C and Humidity in between 35RH to 65RH).
- The used strip and pouch should be disposed off in compliance with the local regulatory requirements.
- Do not use the device after sterility expiry date.
- For External Use only.
- Keep out of reach of children.
- Keep away from sun light.

PRECAUTIONS
 When handling the product care should be taken to avoid damage from handling. Avoid crushing and crimping damage to the product due to application of surgical instruments such as forceps etc.

DIRECTIONS FOR USE


- Do not apply any topical medication to the eye or perform any manipulation of the eye before performing this test.
- Position patient in examining chair with head against a headrest.
- Examination room lights should be dimmed.
- Carefully open strips taking care not to touch the rounded end of the strip.
- While the patient looks up, carefully insert rounded end into the lower temporal lid margin of the eye, making sure the strip fits snugly.
- Wait for 5 minutes, remove the strip and record the length of the moistened area.
- The product should be stored at Temperature 10°-40°C and Humidity 35-65 RH.
- Mishandling or improper transport can cause damage to the device or packaging.

DEVICE LIFE
 The expiration date of the product is the sterility expiration date. The product should not be use after the indicated sterility expiration date.

RETURN OF DAMAGED PRODUCT
 Return the product in its original packing identified by the batch number, purchase information, your reference and reason for return. Please contact your local distributor office regarding product return/exchange.

DISPOSAL OF THE USED DEVICE
 The used device should be disposed off in compliance with the local regulatory requirements.

REPORTING
 Adverse events and/or potentially sight-threatening complications that may reasonably be regarded as product related and that were not previously expected in nature, severity or incidence must be report to Tarun Enterprises.



ACCURACY
 The printed scale of a Schirmer strip is a mm scale from 00 mm to 35 mm with a least count of 1mm. The accuracy of the complete scale (0 to 35)mm is ± 1mm which implies an accuracy of ± 0.028mm between two graduated mm lines.

110mm
BACK

Page 2
Page 3