OPHTHALMIC CORNEAL TREPHINE

INSTRUCTION FOR USE

Intended use:

A disposable sterile circular blade used to take out a circular comeal button of predetermined diameter from the donor or recipient cornea in a comeal transplant (Keratoplasty) procedure.

Mode of supply:

EO gas sterilized single use circular blade of different diameters mounted on an ABS plastic handle (see through for proper centering of comea). Each piece is individually packed and sealed in a pouch before sterilization. The pouch is then packed in a printed outer cardboard box and labeled.

Directions for use:

- 1. Take the sterile trophine of required size out of the box and open the outer pouch (BY AN UNSTERILE PERSON) and drop the inner sealed blister pack on to a sterile
- surface without touching it.

 2. Open the blister and take out the trephine from the holder taking care not to touch
- the blade to any surface.

 3. Take out the corneal button either by using an endothelial punch of directly from the recipient comea manually.

 4. Put back the used trephine in the blister and dispose it with waste for incineration.

Precautions and warning:

- Do not use any of the components if the sterile package is damaged.
 This device must be used only by physicians who have received proper training.
 Do not use after expiry date mentioned on the package label.

TARUN ENTERPRISES

8/8, Strachy Road, Allahabad - 211001 U.P. INDIA e-mail: info@optitecheyecare.com



REF TE/ART/2014-15/01

L. Rev. No. 05 Dated 22nd May 2020